IFU-OTH-0005

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Revisão / Revision: 0  
Data / Date: 29/09/2023

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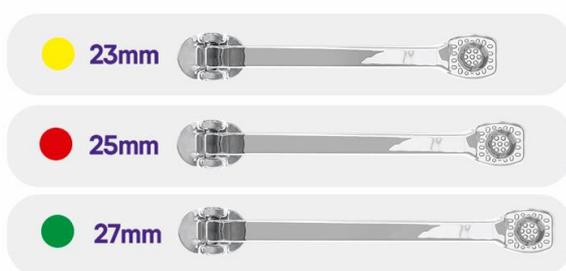
**Instructions for Use**

**Technical name:** Molar Distalizer

**Product name:** CLASS II CORRECTOR - ICERAM CLEAR DISTALIZER

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**Product Description**

The Iceram Distalizer is a device made of a laminated, arched rectangular structure made of polyamide, which adapts at its to accessories similar to a button on the mesial side and a metal tube on the distal side. It is available in 3 sizes: 23 mm, 25 mm e 27 mm. Its main characteristics are that it is easy to install, universal and can be used on the right or left side of the arch, aesthetic and comfortable.

It is a fixed appliance used in the treatment of symmetrical or asymmetrical Class II, for the distalization of molars, premolars and maxillary canines. It is well accepted by patients because it is simple to use and has a discreet apparatus, without the need for accessories bonded to all the teeth. It also allows the posterior block

to be distalized independently and in a straight line, without the resistance of the curved anterior segment, which speeds up anteroposterior correction and eliminates collateral deflection forces.

The device should be fixed mesially on the canine and distally on the buccal surface of the first molar, as its arched shape allows it to adapt properly to the buccal surfaces of the teeth, respecting the anatomy and therefore not disturbing the patient.

### **Intended use**

The Iceram Distalizer is designed to be used in the treatment of Class II malocclusion, making it possible to correct the posterior segment by distal tipping and molar rotation. To apply the necessary force, it must be used in conjunction with Class II elastics. They can be used with fixed appliances in the lower arch or with aligners. The fixation point of the elastics should be on the button bonded to the canine and on a metal or ceramic button or tube bonded to the buccal surface of the lower first molar.

For fixed appliances in the lower arch, it is recommended to reinforce the anchorage with a lingual arch. When associating with the aligner in the lower arch, it should be cut out in the area where the aesthetic button is bonded, thus guaranteeing the adaptation of the aligner plate.

### **How to use:**

#### **1- Installation of Iceram Distalizer:**

Use involves the following steps:

- 1) Measurement of the distance between the center of the buccal aspect of the maxillary canine and the mesial third of the maxillary first molar;
- 2) Selection of the appropriate length, which can be 23 mm, 25 mm or 27 mm;
- 3) Relative isolation with a mouth opener and cotton rollers;
- 4) Prophylaxis with Robinson's brush and pumice stone;
- 5) Rinsing with water and drying;
- 6) Acid conditioning;
- 7) Rinsing with plenty of water and drying;
- 8) Application of adhesive;
- 9) Photopolymerization;
- 10) Application of a layer of resin to the base of the button and tube;
- 11) Positioning the device;
- 12) Removal of excess resin;
- 13) Light-curing;
- 14) Use of a Class II elastic band, either ¼" medium or 3/16". It is recommended to start with a force of 170g in the first month and increase to 230g until the molar relationship is corrected.

## **2- Removing the Distalizer:**

In order to remove the Distalizer, straight or angled bracket removal pliers are used, depending on adaptation and professional preference, using traction and shear force.

### **Composition:**

Hinge - AISI 630 - 17 4 ph 30%

Rod - Grilamid TR90 LX Natural 70%

### **Storage Conditions:**

This product does not require temperature, humidity or light control during storage. The recommendation is to keep it in a clean, dry place.

The manufacturer recommends the single use of this product. It should be disposed of correctly in accordance with the information contained in the Manual of Dental Services Risk Prevention and Control, Technology in Health Services, 2006 (available on the ANVISA website).

### **Warnings and Symbology:**

- It is essential that only the specialist installs the product on the patient, since in addition to the clinical examination, the professional must have the necessary knowledge to make a systematic description of the patient's dentofacial features, list the priority problems, describe possible solutions for each of the problems, considering the possible favorable or unfavorable interactions for each of the proposed attempts.
- It is also the orthodontist's duty to determine the maximum expected gain from orthodontic therapy and the minimum loss, or to calculate the benefit versus risk ratio for each patient, with both risk and benefit including psychosomatic factors in order to determine the best management for each patient.
- Patients should be instructed in oral hygiene, maintenance, and food care. The following should be avoided: ice, coconut, hard and/or sticky sweets, caramels, nougats, jellybeans, corn, popcorn and sugar cane. Biting hard objects such as pens and pencils has the same effect.
- Advise the patient **NOT TO CHEW OR BITE HARD FOOD** during orthodontic treatment, to prevent premature detachment.
- This product should not be used in an MRI environment, as it may cause interference in the images during the examination. It is up to the orthodontic professional to warn the patient of this contraindication and arrange for the appliance to be removed if necessary.

**IFU-OTH-0005**

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Revisão / Revision: **0**  
Data / Date: 29/09/2023

- The high concentration of bacteria and fungi present in the mouth can accelerate the corrosion of metal orthodontic appliances. The pH of the oral environment also has a major influence on the corrosive process, so a diet rich in acidic foods, such as excessive intake of soft drinks with acidic pHs, should be avoided as much as possible during treatment.
- Do not touch the parts with ultrasonic tips: this can cause fractures and detachments.
- The product is supplied in non-sterile condition, however it has proven biocompatibility and no cleaning, disinfection or sterilization measures are required, provided the packaging has not been tampered with. If the product comes into contact with surfaces or substances other than those intended for use, it is recommended that it be discarded.
- The product must not be reused or reprocessed.
- Do not use the product if its packaging is damaged or open, in order to minimize possible dangers related to the transmission of microorganisms or possible product counterfeiting.
- It is the responsibility of qualified professionals to advise their patients of the risk of aspiration and swallowing in the event of breakage/ detachment of parts, not to underestimate the possible immediate or delayed complications resulting from this type of accident and to adequately inform the patient and/or their legal guardians of the risks inherent in the event and the measures to be taken.
- This product should only be used by qualified orthodontic professionals. Use by people without the necessary technical knowledge could lead to unwanted tooth movement, bone loss, tooth loss, fenestration, gingival recession or root dehiscence.
- It is the responsibility of the qualified professional to advise the patient on correct oral hygiene, in order to avoid the appearance of plaque and tartar or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis. Considering that the product is used in contact with mucous membranes and body fluids, it is recommended that professionals apply good clinical practices for the disposal of medical products in accordance with current health regulations.

**ATTENTION: Orthometric cannot be held responsible for the results obtained by applying the product without the necessary precautions or failing to observe the warnings.**

### **Contraindications**

Orthometric only supplies its products to qualified professionals. It is the orthodontist's sole responsibility to

**IFU-OTH-0005**

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Revisão / Revision: **0**  
Data / Date: 29/09/2023

identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene.
- Patients unable to cooperate with treatment.
- Patients with an oral environment conducive to the demineralization of tooth enamel.

	Do not re-use		Caution
	Consult instructions for use		Non-sterile
	Medical device		Contains Nickel
	Catalogue number		Date of manufacture
	Batch code		Keep away from sunlight
	Manufacturer		Do not use if package is damaged
	Use-by date		Keep dry