IFU-OTH-0013

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Revisão / Revision: 1
Data / Date: 07/04/2025

**INSTRUCTIONS FOR
USE**

Technical Name: ORTHOMETRIC ELASTOMERICS

Product Name: ORTHOMETRIC ELASTOMERICS

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- Telemóvel: +351 967 417 654

Product description

The modular elastics in chain (short, medium and long) are made of TPU polymer (Thermoplastic Polyurethane) and have a design that establishes a good adaptation to the tooth movement elements, providing the patient with comfort, quality, safety and reliability.

When used together with brackets, tubes and accessories fixed on the surface of the tooth, the elastics aim to help in the fixation or generation of forces, contributing to establish the mechanical-biological system, and in this way make it possible to apply force to the teeth, which results in induced tooth movement, correcting malocclusion.

Intended use

Elastomeric chains are thermoplastic polyurethane-based materials that have the property of returning to their original shape after substantial deformation. They are widely used to promote the application of force that results in tooth movement, considering the sagittal, vertical and transverse planes. They come in 3 sizes according to the distance between them: short, medium and long. Current elastics can therefore be used in the following clinical situations:

1) Closing spaces resulting from premolar extractions, retraction of anterior teeth, mesialization of posterior

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teeth or a combination of both;

- 2) Closing generalized diastemas in dental arches;
- 3) For torque composition in the correction of dental gyroversions;
- 4) In correcting deviations of the dental midlines;
- 5) For partial retraction of canines; and
- 6) In the traction of dental elements.

Indication for use

Chain elastics are stretched to 50% of their original size and are used to provide the necessary force for tooth movement. They are specifically useful for closing gaps in dental arches resulting from generalized diastemas or premolar extractions.

How to use:

For proper use of chain elastics it is recommended to:

- 1) Measure the distance from one application point to the other;
- 2) Quantify the number of links that adapt to this distance;
- 3) Select whether the elastomeric chain to be used will be short, medium or long;
- 4) Cut the segment of elastic chain according to the distance to be applied;
- 5) Attach a link to one of the ends and stretch the elastic, fitting it to another application point;
- 6) Stretch the elastic to 50% of its size;
- 7) Avoid grasping the elastic between the active ends of the needle holder, as this could cause the elastic to break or the link to deform;
- 8) Discard the last link at each end, as it may be irregular when the elastic is cut; and
- 9) Advise the patient on hygiene care, as elastomeric chains can encourage the accumulation of oral biofilm and induce the occurrence of gingivitis;
- 10) The current elastic should be replaced monthly, until the proposed objective is achieved or according to the professional's plan.

Elastics should only be used by professionals qualified to practice orthodontics and follow the recommendations mentioned above. It is the sole responsibility of the professional to analyze for which clinical situations elastics can be used, and when the use of elastomeric chains is not indicated, or even to identify when the use of elastics should be discontinued.

It should be emphasized that when using elastics, the activation criterion of a maximum of 50% stretching of the original length of the elastic must be obeyed, as this could result in a force greater than desirable for tooth movement, or induce greater deformation than usually occurs when using chain elastics.

Some warnings can be listed when using chain elastics:

- 1) Avoid using elastics whose packaging has been tampered with, to avoid the risk of cross-contamination between professionals and patients;
- 2) Make sure the elastic is original and within its expiration date;
- 3) Elastics should be stored in a suitable environment, away from heat, dry environments and sunlight;
- 4) Advise the patient on proper hygiene;
- 5) Allergic reactions can occur when using elastics and the patient should be advised to report any such occurrence.

Composition:

<i>Material</i>	<i>Concentration</i>
<i>Thermopolyurethane</i>	<i><97%</i>
<i>Masterbatch</i>	<i>>3%</i>

Storage Conditions:

In intact and closed packaging.

This product must be stored in a clean, dry environment, away from heat sources and sunlight, as inadequate conservation of the elastics may compromise the properties of the material, impairing the functioning of the product..

The manufacturer recommends the single use of this product. It should be disposed of correctly in accordance with the information contained in the Manual of Dental Services Risk Prevention and Control, Technology in Health Services, 2006 (available on the ANVISA website).

Warnings and Symbology:

- It is essential that only the specialist installs the product on the patient, since in addition to the clinical examination, the professional must have the necessary knowledge to make a systematic description of the patient's dentofacial features, list the priority problems, describe possible solutions for each of the problems, considering the possible favorable or unfavorable interactions for each of the proposed attempts.
- It is also the orthodontist's duty to determine the maximum expected gain from orthodontic therapy and the minimum loss, or to calculate the benefit versus risk ratio for each patient, with both risk and benefit including psychosomatic factors to determine the best management for each patient.
- Patients should be instructed in oral hygiene, maintenance, and food care. The following should be avoided: ice, coconut, hard and/or sticky sweets, caramels, nougats, jellybeans, corn, popcorn, and sugar cane. Biting hard objects such as pens and pencils has the same effect. Chewing gum and toffee often loosen the bands or rings that have been cemented in place.
- The product is supplied in non-sterile condition, however it has proven biocompatibility and no cleaning, disinfection or sterilization measures are required, provided the packaging has not been tampered. If the product comes into contact with surfaces or substances other than those intended for use, it is recommended that it be discarded.

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Revisão / Revision: 1
Data / Date: 07/04/2025

- The product must not be reused or reprocessed. Do not use the product if its packaging is damaged or open, to minimize possible dangers related to the transmission of microorganisms or possible product counterfeiting.
- It is the responsibility of qualified professionals to advise their patients of the risk of aspiration and swallowing in the event of breakage/ detachment of parts, do not to underestimate the possible immediate or delayed complications resulting from this type of accident and to adequately inform the patient and/or their legal guardians of the risks inherent in the event and the measures to be taken.
- This product should only be used by qualified orthodontic professionals. The use by people without the necessary technical knowledge could lead to unwanted tooth movement, bone loss, tooth loss, fenestration, gingival recession, or root dehiscence.
- It is the responsibility of the qualified professional to advise the patient on correct oral hygiene, in order to avoid the appearance of plaque and tartar or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis. Considering that the product is used in contact with mucous membranes and body fluids, it is recommended that professionals apply good clinical practices for the disposal of medical products in accordance with current health regulations.

ATTENTION: Orthometric cannot be held responsible for the results obtained by applying the product without the necessary precautions or failing to observe the warnings.

Contraindications

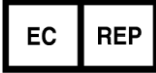
Orthometric only supplies its products to qualified professionals. It is the orthodontist's sole responsibility to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene.
- Patients unable to cooperate with treatment.
- Patients with a disease or limitation that makes the treatment difficult.

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Revisão / Revision: 1
Data / Date: 07/04/2025

	Do not re-use		Authorized representative in the European Community
	Consult instructions for use		European conformity
	Medical device		Caution
	Catalogue number		Non-sterile
	Batch code		Date of manufacture
	Manufacturer		Keep away from sunlight
	Use-by date		Do not use if package is damaged
	Keep dry		