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Revisão / Revision: 1  
Data / Date: 31/03/2025

## INSTRUCTIONS FOR USE

**Technical Name:** ORTHOMETRIC ELASTOMERICS

**Product Name:** ORTHOMETRIC ELASTOMERICS

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### Product description

The modular elastics are made of TPU polymer (Thermoplastic Polyurethane) and have a design that establishes a good adaptation to the tooth movement elements, providing the patient with comfort, quality, safety and reliability.

The ligatures present in the module aim to assist in fixing the orthodontic arch or wire in the bracket slot, contributing to establishing the mechanical-biological system, and thus enabling the application of force to the teeth, resulting in induced tooth movement to correct the malocclusion.

### Intended Use

The elastic has the property of returning to their original shape, with a certain deformation of the material, and are stretched around the bracket fins, promoting the fixation of the orthodontic arch to the accessory channel, preventing its displacement, and thus enabling the application of the force necessary for tooth movement.

### How to Use

For the proper use of modular elastic, it is recommended that:

- 1) Separating the elastic rings from the support structure;
- 2) Application of the elastic with the Sputnik instrument, which has two active tips, one tapered and the other curved, forming a handle or by means of a needle holder;
- 3) The elastic is stretched and adjusted on the curved tip of the Sputnik or grasped with the needle holder;
- 4) The elastic is guided to the bracket and a rotating movement from cervical to occlusal is promoted;
- 5) Check that the elastic has adapted perfectly to the bracket, involving all the flaps;
- 6) Advise the patient on hygiene precautions, as the elastic may encourage the accumulation of food residue

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and cause some irritation to the soft tissue, or even cause stains if the problem persists routinely.

### **Instructions for Removing the Elastics**

To remove the elastic, use the tapered active tip of the Sputnik instrument and disengage the elastic from the bracket fins. The elastic must be disposed of properly as it contains food residues and micro-organisms, avoiding contamination of the professional and assistant.

### **Some warnings can be listed when using elastics**

- 1) Avoid using elastic whose packaging has been tampered with, to avoid the risk of cross-contamination between professionals and patients;
- 2) Make sure the elastic is original and within its expiration date;
- 3) Elastics should be stored in a suitable environment, away from heat, dry environments and sunlight;
- 4) Advise the patient on proper hygiene;
- 5) Allergic reactions can occur when using elastics and the patient should be instructed to report this;
- 6) The professional should warn the patient of the risks of using elastic: tooth sensitivity; possible allergic reaction and tooth decalcification when the area is not properly cleaned; and
- 7) Any undesirable effect resulting from the use of elastic is considered a professional responsibility.

### **Composition:**

<i>Material</i>	<i>Concentration</i>
<i>Thermopolyurethane</i>	<i>&lt;97%</i>
<i>Masterbatch</i>	<i>&gt;3%</i>

### **Storage Conditions:**

In intact and closed packaging.

This product must be stored in a clean, dry environment, away from heat sources and sunlight, as inadequate conservation of the elastics may compromise the properties of the material, impairing the functioning of the product.

The manufacturer recommends the single use of this product. It should be disposed of correctly in accordance with the information contained in the Manual of Dental Services Risk Prevention and Control, Technology in Health Services, 2006 (available on the ANVISA website).

### **Warnings and Symbology:**

- It is essential that only the specialist installs the product on the patient, since in addition to the clinical examination, the professional must have the necessary knowledge to make a systematic description of the patient's dentofacial features, list the priority problems, describe possible solutions for each of the problems, considering the possible favorable or unfavorable interactions for each of the proposed attempts.

**IFU-OTH-0015**

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Revisão / Revision: 1  
Data / Date: 31/03/2025

- It is also the orthodontist's duty to determine the maximum expected gain from orthodontic therapy and the minimum loss, or to calculate the benefit versus risk ratio for each patient, with both risk and benefit including psychosomatic factors to determine the best management for each patient.
- Patients should be instructed in oral hygiene, maintenance, and food care. The following should be avoided: ice, coconut, hard and/or sticky sweets, caramels, nougats, jellybeans, corn, popcorn, and sugar cane. Biting hard objects such as pens and pencils has the same effect. Chewing gum and toffee often loosen the bands or rings that have been cemented in place.
- The product is supplied in non-sterile condition, however it has proven biocompatibility and no cleaning, disinfection or sterilization measures are required, provided the packaging has not been tampered. If the product comes into contact with surfaces or substances other than those intended for use, it is recommended that it be discarded.
- The product must not be reused or reprocessed. Do not use the product if its packaging is damaged or open, to minimize possible dangers related to the transmission of microorganisms or possible product counterfeiting.
- It is the responsibility of qualified professionals to advise their patients of the risk of aspiration and swallowing in the event of breakage/ detachment of parts, do not to underestimate the possible immediate or delayed complications resulting from this type of accident and to adequately inform the patient and/or their legal guardians of the risks inherent in the event and the measures to be taken.
- This product should only be used by qualified orthodontic professionals. The use by people without the necessary technical knowledge could lead to unwanted tooth movement, bone loss, tooth loss, fenestration, gingival recession, or root dehiscence.
- It is the responsibility of the qualified professional to advise the patient on correct oral hygiene, in order to avoid the appearance of plaque and tartar or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis. Considering that the product is used in contact with mucous membranes and body fluids, it is recommended that professionals apply good clinical practices for the disposal of medical products in accordance with current health regulations.

**ATTENTION: Orthometric cannot be held responsible for the results obtained by applying the product without the necessary precautions or failing to observe the warnings.**

**Contraindications**

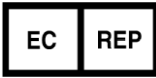
Orthometric only supplies its products to qualified professionals. It is the orthodontist's sole responsibility to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene.
- Patients unable to cooperate with treatment.
- Patients with a disease or limitation that makes the treatment difficult.

**IFU-OTH-0015**

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Revisão / Revision: 1  
Data / Date: 31/03/2025

	Do not re-use		Authorized representative in the European Community
	Consult instructions for use		European conformity
	Medical device		Caution
	Catalogue number		Non-sterile
	Batch code		Date of manufacture
	Manufacturer		Keep away from sunlight
	Use-by date		Do not use if package is damaged
	Keep dry		